

How to use your rapid lateral flow test

Remember to **always read the instructions** that come with the test kit

	COVID-19 Self-Test (Rapid Antigen Test)	Orient Gene	ACON Flowflex	MP Biomedicals
Swabbing process	 	 	 	
Swab in tube	15 secs	6 times	30 secs + rotate 5 times	rotate 2-5 times
Resting time	None	1 min	None	1 min
Drops on device	2	4	4	3
Wait time	 IMPORTANT Do not leave the test to develop for longer than 30 minutes as this will make the result void.	 IMPORTANT Wait for 15 minutes to read the result. Do not read the result after 20 minutes .	 IMPORTANT Wait for 15 minutes to read the result. Do not read the result after 30 minutes .	 IMPORTANT 15-20 minutes (results after 20 minutes may not be accurate)

Now report your result:

Type in:
<https://portal.spft.nhs.uk/covid-19>
to get to the reporting page

Report **all** results whether they are **positive, negative** or **void**

You will need:

- the code on the test cassette

You've only completed your test once you've reported the result

Have you had your **Covid-19 vaccination** yet? If not, please visit: nhs.uk/conditions/coronavirus-covid-19/coronavirus-vaccination/